EpiTrain[®]

ELLENBOGENBANDAGE

ELBOW SUPPORT
ORTHÈSE DE COUDE
TUTORE PER GOMITO

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Dear customer,
thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

EpiTrain is a medical device. The elbow support is a support for the relief and stabilization of the elbow joint.

Indications

- Post-traumatic or post-operative irritation, e.g. epicondylitis
- Osteoarthritis / arthritis (joint wear / joint inflammation)
- Tendinopathy (degenerative tendon condition)
- Tendomyopathy (muscle-tendon condition)
- (Relapse) prevention

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

The benefits of this product are felt particularly during physical activity.

- Please remove the support during longer periods of rest.
- Once you have been prescribed the EpiTrain, only use it according to your indication and any additional instructions given by a medical specialist*. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- For this reason, check the fit of the product if you feel unpleasant pressure and consult the specialist in this regard or for general questions about handling.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.
- Caution: When removing the product, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.

Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present; likewise, any raised scars with swelling, redness, and excessive heat build-up
- Abnormal sensation in the arm
- Impaired circulation in the arm
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

Application instructions

Putting on the EpiTrain

For easy donning, fold the top part of the support outward so that the cutouts for the internal massage pads become visible . Then grab the support near the pad pocket 

pull it over the elbow. Make sure that the outer bony protrusion of the elbow (epicondyle) is positioned in the cut out of the pad, just the same as the inner bony protrusion of the elbow . Then fold the upper edge of the support back into the original position .

Taking off the EpiTrain

For easy doffing, grab the support at the bottom edge near the pad pocket and pull it toward your hand.

Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be claimed before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the EpiTrain have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Styrene-ethylene / butylene-styrene block copolymer (TPS-SEBS), Spandex (EL), Cotton (CO), Polyurethane (PUR), Polyester (PES)

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Bauerfeind USA, Inc.
BAUERFEIND AG