**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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Polyamid (PA), Aluminium (AlMgSi1), Polyurethan (PUR), Polyoxymethylen (POM), Baumwolle (CO), Polyester (PES), Elastan (EL)

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en english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

## Intended purpose

The GenuLoc is a medical device. It is an orthosis<sup>1</sup> for immobilizing the knee joint – either with the leg extended or at a preset flexion angle.

**Fitting of the GenuLoc may only be carried out by trained specialists<sup>2</sup>, otherwise product liability cannot be assumed.**

## Indications

- Pre-operative, post-operative, post-traumatic immobilization
- Non-surgical treatment of patellar fractures

## Risks of using this product

### Important information

- Correct usage / fitting is assumed. Any therapeutic medical aids – supports and orthoses – applied externally to the body can, if tightened excessively, lead to local pressure symptoms or, rarely, constrict blood vessels or nerves. If you experience this, it can be helpful to loosen the straps slightly and immediately consult your physician and trained specialist.
- The knee orthosis GenuLoc is a prescribed product that should be applied under a physician’s guidance. GenuLoc should only be worn in accordance with these instructions for use and for the listed areas of application.
- Product liability cannot be assumed in the event of inappropriate use.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby excluding any product liability.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor. Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Combining the support with other products (e.g. compression stockings) must be discussed beforehand with your physician.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.

## Contraindications

Side effects involving the body as a whole have not been reported to date.

- No clinically significant hypersensitive reactions have been reported to date. Generally, treatment with a knee immobilization splint is prescribed by a physician. If this has not occurred and you have any of the following conditions, consult your physician before fitting and wearing the product:
  - Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. in case of any raised scars with swelling, redness or excessive heat build-up.
  - Severe varicose veins (varicosis), particularly with impaired return flow of blood to the heart.
  - Impaired lymphatic drainage – including unspecified soft tissue swelling located in distal direction of the fitted aid.
  - Impaired sensation and circulatory disorders in the legs, e.g. diabetes mellitus.

## Application instructions

GenuLoc is available in one universal size and can be adjusted individually. For this reason, size determination is not carried out.

### **Before putting on the orthosis for the first time:**

- Before using the product, the specialist must check for the need of thrombosis prophylaxis.
- The preset bending angle must be checked by the specialist and adjusted if necessary.
- Before putting on the system, the circumference of the orthosis is adjusted using the lateral Velcro straps. The side parts must be attached in the same way on both sides (Image 1 / 2).
- If the use of a pad behind the knee is required (please ask your physician about this), attach this to the Velcro area provided on the inside of the orthosis before putting on the product.

## Putting on the orthosis

- When positioning the system on the leg, make sure that the distance from the middle straps to the middle of the knee caps is uniform (Image 3 / 4).
- To close the system, start with the lower strap near the joint (Image 5).
- Then close the upper strap near the joint (Image 6).
- To put the orthosis on completely, close the lower strap away from the joint and then the upper strap away from the joint (Image 7 / 8).
- To firmly fix the orthosis in place, first evenly pull tight the two lower straps running in opposite directions. Proceed the same way with the two upper straps (Image 9).

### **Taking off the orthosis**

- When taking off the GenuLoc, the straps are opened in the reverse order (Image 9, 8, 7, 6, 5). For your own safety, carry this out while sitting.

## Cleaning instructions

- Please only wash the orthosis by hand at 30 °C using a mild detergent. All components can stay on the orthosis during washing. Leave the orthosis to air dry.
- Regular care guarantees optimum effectiveness of the product. Also observe the instructions on the label behind the pad at the back of the knee.
- Never expose the product to direct heat / cold as this may impair its effectiveness.
- If you have any complaints, please contact your retailer. Only clean goods are handled.

## Part of the body this product is used for

In accordance with the indications (knee). See purpose.

## Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

## Technical specifications / parameters

The GenuLoc is a stabilizing orthosis for the knee joint. It consists of supporting material, four stabilizing stays and four straps.

## Notes on reuse

This product is intended to be fitted to one patient only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuLoc have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

## Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

Polyamide (PA), Aluminium (AlMgSi1), Polyurethane (PUR), Polyoxymethylene (POM), Cotton (CO), Polyester (PES), Spandex (EL)

MD – Medical Device

UDI – DataMatrix Unique Device Identifier UDI

Version: 2023-10

<sup>1</sup> Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

<sup>2</sup> A specialist is any person who is authorized, according to their state’s regulations, to fit and provide instructions on the use of supports and orthoses.

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