AUSTRIA
 Bauerfeind Ges.m.b.H.
 Hainburger Straße 33
 1030 Wien
 P +43 (0) 800 4430-130
 F +43 (0) 800 4430-131
 E info@bauerfeind.at

BENELUX
 Bauerfeind Benelux B.V.
 Waarderveldweg 1
 2031 BK Haarlem
 THE NETHERLANDS
 P +31 (0) 23 531-9427
 F +31 (0) 23 532-1970
 E info@bauerfeind.nl

BELGIUM
 P +32 (0) 2 527-4060
 F +32 (0) 2 792-5345
 E info@bauerfeind.be

BOSNIA AND HERZEGOVINA
 Bauerfeind d.o.o.
 Meše Selmovića 19
 71000 Sarajevo
 P +387 (0) 33 710-100
 F +387 (0) 33 619-422
 E info@bauerfeind.ba

CROATIA
 Bauerfeind d.o.o.
 Goleška 20
 10020 Zagreb
 P +385 (0) 1 6542-855
 F +385 (0) 1 6542-860
 E info@bauerfeind.hr

FRANCE
 Bauerfeind France S.A.R.L.
 B.P. 59258
 95957 Roissy CDG Cedex
 P +33 (0) 1 4863-2896
 F +33 (0) 1 4863-2963
 E info@bauerfeind.fr

ITALY
 Bauerfeind Italia Srl
 Via Cornaggia 58
 20092 Cinisello Balsamo (MI)
 P +39 02 8977 6310
 F +39 02 8977 5900
 E info@bauerfeind.it

REPUBLIC OF NORTH MACEDONIA
 Bauerfeind Dooel Skopje
 50 Divizija 24 a
 1000 Skopje
 P +389 (0) 2 3179-002
 F +389 (0) 2 3179-004
 E info@bauerfeind.mk

SERBIA
 Bauerfeind d.o.o.
 102 Omladinskih brigada
 11070 Novi Beograd
 P +381 (0) 11 2287-050
 F +381 (0) 11 2287-052
 E info@bauerfeind.rs

SINGAPORE
 Bauerfeind Singapore Pte Ltd.
 Blk 41 Cambridge Road #01-21
 Singapore 210041
 P +65 6396-3497
 F +65 6295-5062
 E info@bauerfeind.com.sg

SLOVENIA
 Bauerfeind d.o.o.
 Dolenjska cesta 242 b
 1000 Ljubljana
 P +386 (0) 1 4272-941
 F +386 (0) 1 4272-951
 E info@bauerfeind.si

SPAIN
 Bauerfeind Ibérica, S.A.
 C / San Vicente Mártir,
 nº 71 - 4º - 7ª
 46007 Valencia
 P +34 96 385-6633
 F +34 96 385-6699
 E info@bauerfeind.es

SWITZERLAND
 Bauerfeind AG
 Vorderer Böde 5
 5452 Oberrohrdorf
 P +41 (0) 56 485-8242
 F +41 (0) 56 485-8259
 E info@bauerfeind.ch

UNITED ARAB EMIRATES
 Bauerfeind Middle East FZ LLC
 Dubai Healthcare City
 Building 40, P.O. Box 505116
 Dubai
 P +971 4 4335-684
 F +971 4 4370-344
 E info@bauerfeind.ae

UNITED KINGDOM
 Bauerfeind UK
 85 Tottenham Court Road
 London
 W1T 4TQ
 P +44 (0) 121 446-5353
 F +44 (0) 121 446-5454
 E info@bauerfeind.co.uk

USA
 Bauerfeind USA, Inc.
 75 14th St NE
 Suite 2350
 Atlanta, GA 30309
 P +1 800 423-3405
 P +1 404 201-7800
 F +1 404 201-7839
 E info@bauerfeindusa.com

86°F

BAUERFEIND AG
 Triebeser Straße 16
 07937 Zeulenroda-Triebes
 Germany
 P +49 (0) 36628 66-40 00
 F +49 (0) 36628 66-44 99
 E info@bauerfeind.com

BAUERFEIND.COM

MalleoTrain® Plus

Sprunggelenkbandage
 Ankle support
 Chevillère
 Tutore per la caviglia
 Vendaje para el tobillo

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and closely observe the specifications in these instructions for use and the instructions given by the specialist. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

MalleoTrain Plus is a medical device. It is a support used to provide increased stabilization and relief of the ankle thanks to a functional strap system.

Indications

- Chronic, post-traumatic or post-operative irritation
- Postoperative recovery
- Early functional treatment of capsular ligament injuries in the upper / lower ankle
- Ligament insufficiency / instability
- Supination prophylaxis, particularly during increased activity

Risks of using this product

The benefits of this product are felt particularly during physical activity.

- Please remove the support during longer periods of rest.
- Once you have been prescribed the MalleoTrain Plus, only use it according to your indication and any additional instructions given by a medical specialist*. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases,

constrict the underlying blood vessels or nerves.

- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.
- Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Furthermore, proceed with extreme caution, as your ability to stand and walk is also heavily impacted.
- Check the specific ground characteristics and ensure the ground is not slippery whenever walking, standing or otherwise stressing the affected part of the body. If the ground is smooth, there is an acute risk of slipping and thus injury!

Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up.
2. Impaired blood flow in the leg / foot (e.g. with diabetes mellitus)
3. Impaired lymphatic drainage and soft tissue swelling of unknown origin.

Application instructions

Putting on the MalleoTrain Plus

- Fold the top part of the support outward until you can reach the heel. Pull the support over the foot until it reaches the heel.
- Then unfold the turned-over part of the support and pull it up until the pads are around the ankle.
- The start of the strap is located on the inside of the foot.  When winding the strap, make sure that it sits without folds. To ensure the optimum fit of the strap, your lower leg must be perpendicular (at a right angle) to the foot during all the following steps.
- Guide the strap under the sole of your foot toward the outside of the foot.  By pulling the strap, you can generate the stabilization that's comfortable for you.
- Place the strap diagonally across the instep and back around the ankle mortise

 and then guide the strap diagonally forward again to the inside of the foot. 

- Pull the strap under the sole of your foot toward the outside of the foot and over the instep  and fasten the end of the strap to the section of the strap underneath it. 

Taking off the MalleoTrain Plus

- Loosen the strap and attach the end of the strap onto another part of the strap. Take hold of the upper edge of the support and turn it down.

Cleaning instructions

Close all the Velcro fastenings. Please use mild detergent and a lingerie bag. Never expose your product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the MalleoTrain Plus have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Styrene-ethylene / butylene-styrene block copolymer (TPS-SEBS), Spandex (EL), Polyurethane (PUR), Cotton (CO), Polyester (PES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2023-06

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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- תחילת הרצועה נמצא בצד הפנימי של כף הרגל. © יש לשים לב שלא ייווצרו קפלים בעת הלבשה. לצורך השגת מנוח מיטבי של הרצועה, השוק שלכם צריכה להיות בניצב לכף הרגל (זווית ישירה) בכל שלב הלבשה הבאים.