 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49(0) 36628 66-35 00
F +49(0) 36628 66-39 99
E info@bauerfeind.com
BAUERFEIND.COM

Rev.14 – 2023-06_103851

ViscoBalance®

Viskoelastische Einlagen
Viscoelastic heel cushions
Talonnettes visco-élastiques
Plantari viscoelastici
Taloneras viscoelásticas

 **MADE**
 **IN**
 **GERMANY**

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and closely observe the specifications in these instructions for use and the instructions given by the specialist. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ViscoBalance is a medical device for the feet. It is a viscoelastic heel cushion to compensate for different leg lengths of 3, 5 and 10 mm. With the heel wedge, the product also offers relief of the Achilles tendon.

Indications

- Difference in leg lengths
- Achillodynia (tendinitis, bursitis subachillea)

Risks of using this product

Important Information

- **ViscoBalance must be fitted for the first use by a trained specialist¹, otherwise we cannot accept any liability for the product.**
- ViscoBalance is a prescribed product that should be applied

under a physician's guidance. ViscoBalance should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.

- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- If there is not enough space inside the shoe for the foot and the selected heel cushion, or if heel cushions are worn in shoes of an incorrect length and width (or shoes with inadequate lateral guidance in the case of open shoe types), problems relating to poor fit may arise (e.g. a feeling of constriction or pressure, rubbing). If the feet are positioned on the edges of the foot orthoses, blisters or calluses may form on the soles of the feet. These may also occur if top covers and / or cushioning material develop holes or creases or break due to mechanical overloading. You must stop wearing heel cushions if they are damaged in this way!
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

- The use of ViscoBalance during sports should be discussed with the medical provider.

Contraindications

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed.

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

Skin disorders / injuries to the relevant part of the body, particularly if inflammation, arterial occlusive vascular disease, or disorders of the lymph system are present. Likewise, any raised scars with swelling, redness, and excessive heat build-up. People with diabetes or rheumatoid foot deformities must seek medical advice before using this product to ensure optimum treatment.

Application instructions

Insert the ViscoBalance heel cushion in the shoe so that it lies flush with the heel section of the shoe.

Product care

We recommend washing ViscoBalance with lukewarm water.

Notes on reuse

This product is intended to be fitted to one patient only.

Places of use

The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ViscoBalance have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and

determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Silicone (SI)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2023-06

- * A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.