 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-35 00
F +49 (0) 36628 66-39 99
E info@bauerfeind.com
BAUERFEIND.COM

ViscoSpot®

Viskoelastische Einlagen
Viscoelastic heel cushions
Talonnettes visco-élastiques
Plantari viscoelastici
Taloneras viscoelásticas

(de) deutsch.....	2	(hr) hrvatski.....	27
(en) english.....	4	(sr) srpski.....	29
(fr) français.....	5	(sl) slovenski.....	31
(nl) nederlands.....	7	(ro) romanian.....	32
(it) italiano.....	9	(el) ελληνικά.....	34
(es) español.....	10	(tr) türkçe.....	36
(pt) português.....	12	(ru) русский.....	38
(sv) svenska.....	14	(lv) latviešu.....	40
(no) norsk.....	16	(li) lietuvių.....	41
(fi) suomi.....	17	(uk) українська.....	43
(da) dansk.....	19	(he) עברית.....	46
(pl) polski.....	21	(ja) 日本語.....	48
(cs) česky.....	22	(ko) 한국어.....	49
(sk) slovensky.....	24	(ar) عربي.....	51
(hu) magyar.....	26	(zh) 中文.....	53

(de) deutsch

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en english

Dear Customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ViscoSpot is a medical device for the feet. It is a viscoelastic heel cushion that provides shock absorption and relief during load peaks. ViscoSpot can be used for redistribution of pressure and pain reduction when treating pathologies in the heel and hindfoot area.

Indications

- Calcaneal apophysitis
- Plantar fasciitis (heel spur)
- Achillodynia
- Osteoarthritis of the ankle
- Osteoarthritis of the knee
- Osteoarthritis of the hip
- Sacroiliac osteoarthritis
- Caudal intervertebral osteoarthritis
- Cement-free endoprosthesis care (in the first 12 months after surgery)

Risks of using this product

Caution*

- ViscoSpot must only be worn according to the specifications in these instructions for use. To ensure the optimum fit of the ViscoSpot, the size required must be determined exactly.
- Only a trained specialist¹ may carry out the (initial) fitting of the support and provide instruction.

- Combination with other products, e.g. for evening out leg lengths, should be discussed beforehand with the physician treating you.
- Should you notice increased symptoms or any unusual changes, please stop using the product and please contact your physician without delay.
- No product liability is accepted in the event of improper use.
- Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- Side effects that affect the entire body have not been reported to date. Correct fitting is assumed.

Contraindications

No clinically relevant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and applied after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation, impaired sensation, arterial occlusive vascular disease, or disorders of the lymph system are present. Likewise, any raised scars with swelling, redness, and excessive heat build-up.

Application instructions

Inserting the ViscoSpot

Use the ViscoSpot in shoes with a stiff heel counter. Discuss the use of the ViscoSpot with the physician treating you beforehand, particularly if you want to use it in cushioned running shoes.

Unless otherwise prescribed by your physician, place both ViscoSpot heel cushions in your shoes so the shape fits snugly into the heel area of the shoes. Please take the right and left versions into account when doing this.

Cleaning instructions

Note: Never expose the ViscoSpot to direct heat (e.g. heater, sunlight, in your car, etc.)! This may cause damage to the material, which can impair the effectiveness of the ViscoSpot.

If necessary, you can clean your ViscoSpot heel cushions with a mild detergent in lukewarm water. Leave the heel cushions to air dry.

Notes on reuse

This product is intended to be fitted to one patient only.

Places of use

The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ViscoSpot have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Silicone (SI)

 – Medical device

 – DataMatrix Unique Device Identifier

Version: 2023-09

*Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

¹A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.