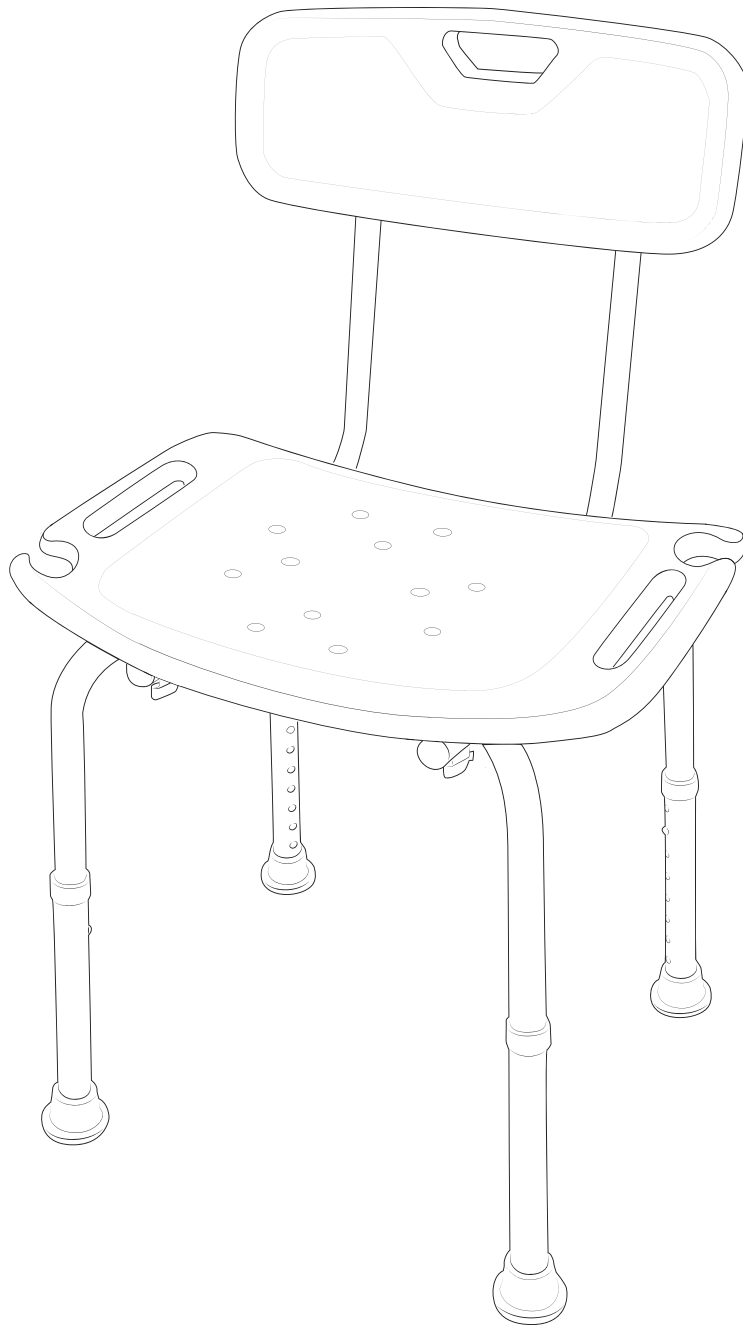
KL4204

General guidelines

- Take the time to thoroughly read the following instruction and utilize the product accordingly.
- Ensure you retain this manual and pass it along when transferring the product.
- Please be aware that this summary may not encompass every detail of all variations and deliberated steps. Feel free to reach out to us for additional information and assistance.

Notes

- The maximum static load capacity is 137 kg (300 lbs), this must not be exceeded.
- Prior to assembly, carefully inspect the components for any damage and strictly follow the provided instructions. Incorrect assembly or damaged components pose a risk of injury.
- Ensure the screws are inserted into the right holes and gradually tighten them.
- Under no circumstances should you use the product if any component is missing or defective.
- Note: Please place the shower stool on a level surface and ensure the base is in full contact with the ground. Avoid sitting only on the edge of the stool. Engaging in dangerous actions such as playing or stooping on the shower stool may cause imbalance.
- When standing up and sitting down, hold both handles of the shower stool firmly.
- Regularly check the firmness of connections and tighten the screws if necessary.
- The shower stool's rubber feet provide stability and grip on wet surfaces. Never remove the rubber feet from the shower stool.
- Always adjust the adjustable feet to the same height. If one or more legs are not level, it can cause instability of the shower stool.
- Never stand on the shower stool or use it as a ladder or climbing tool.
- Avoid exposing the shower stool to direct sunlight and keep it away from sharp objects.
- This product is intended for indoor use by a single individual.
- Only store the shower stool when it is completely dry. If the shower stool is still damp, mould may develop.

Warnings

- To prevent the risk of suffocation, keep all parts and packaging materials (such as films, plastic bags, and foam) away from babies and children.
- While assembling, keep small parts out of the reach of children. Swallowing or inhaling these parts can be fatal.
- Kindly ensure proper usage of the chair. It is strictly prohibited to use it as a climbing tool or sit on the armrests. Children must be supervised by adults when using the product.
- Alterations or modifications to the product without authorization can compromise safety.

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