

KL2108

KM2209

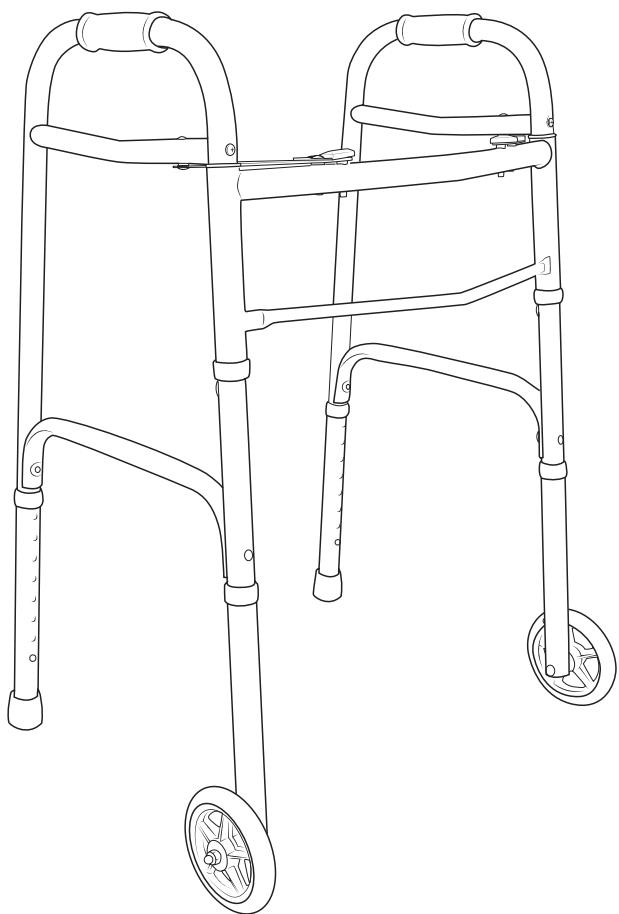EN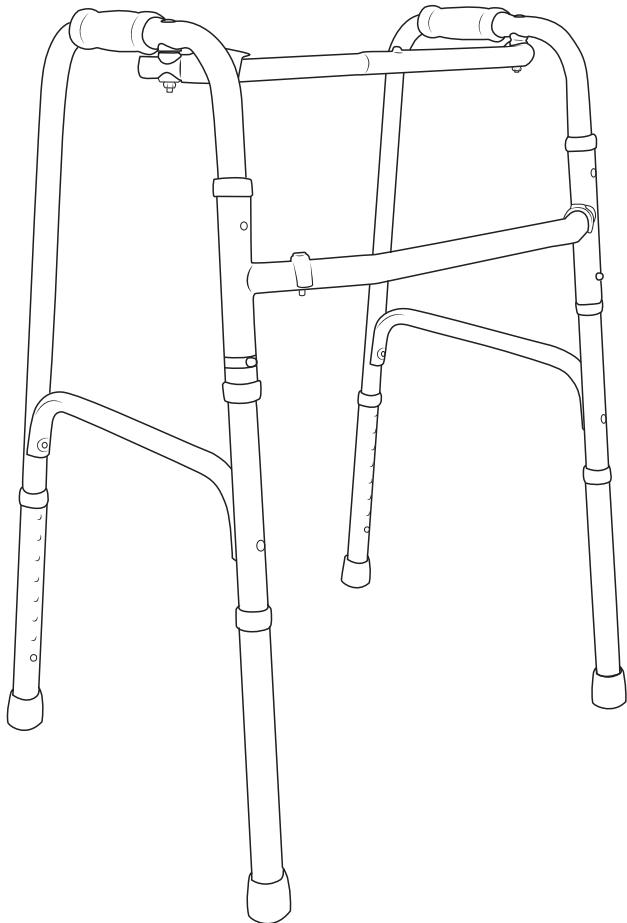

General guidelines

- Take the time to thoroughly read the following instruction and utilize the product accordingly.
- Ensure you retain this manual and pass it along when transferring the product.
- Please be aware that this summary may not encompass every detail of all variations and deliberated steps. Feel free to reach out to us for additional information and assistance.

Notes

- To set the correct height of the walker, keep your arm extended beside your body. The handle of the walker should then be at wrist height.
- The maximum static load capacity is 150 kg (330 lbs), this must not be exceeded.
- Prior to assembly, carefully inspect the components for any damage and strictly follow the provided instructions. Incorrect assembly or damaged components pose a risk of injury.
- When unfolding the walker, make sure the pegs click into the holes and the walker cannot fold inadvertently during use.
- Under no circumstances should you use the product if any component is missing or defective.
- Note: Please place the walker on a level surface and ensure the base is in full contact with the ground. Engaging in dangerous actions such as playing or stooping on the walker may cause imbalance.
- The tips of the walker are made of soft, non-slip material. Do not remove these rubber tips to avoid imbalance and accidents. Replace tips immediately when worn or missing.
- Never stand on the walker or use it as a ladder or climbing tool.
- Avoid exposing the product to direct sunlight and too much water and keep it away from sharp objects.
- This product is intended for indoor and outdoor use by a single individual.
- Only use the walker as instructed by your healthcare provider.

Warnings

- To prevent the risk of suffocation, keep all parts and packaging materials (such as films, plastic bags, and foam) away from babies and children.
- Alterations or modifications to the product without authorization can compromise safety.
- Do not use the walker on stairs unless otherwise instructed by a professional.
- The use of a corroded walker may result in damage to the walker and/or injuries to the user.
- Do not use your walker if you have any handicaps or disabilities or using medications, etc. that would impair or interfere with safe usage of this product.
- Clean the grips with a mild soap detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

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