

Zimmer frame
Gehgestell

| **CR251**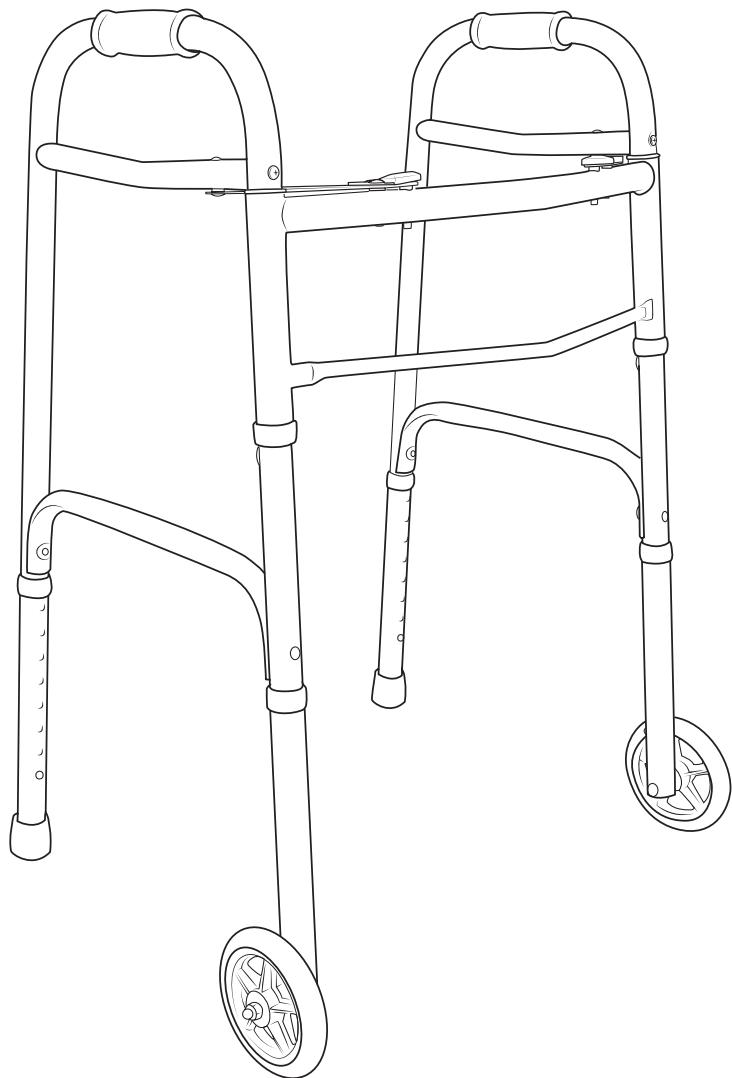


A

x1

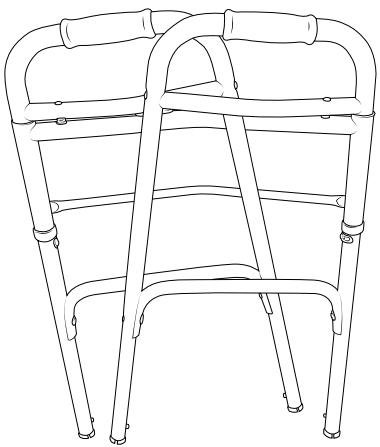
B

x1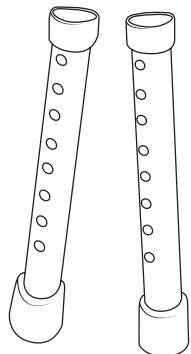


C

x1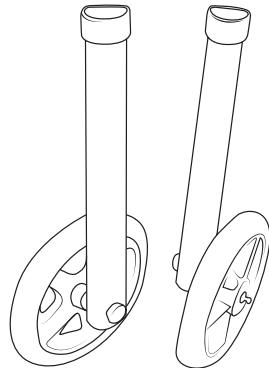


1

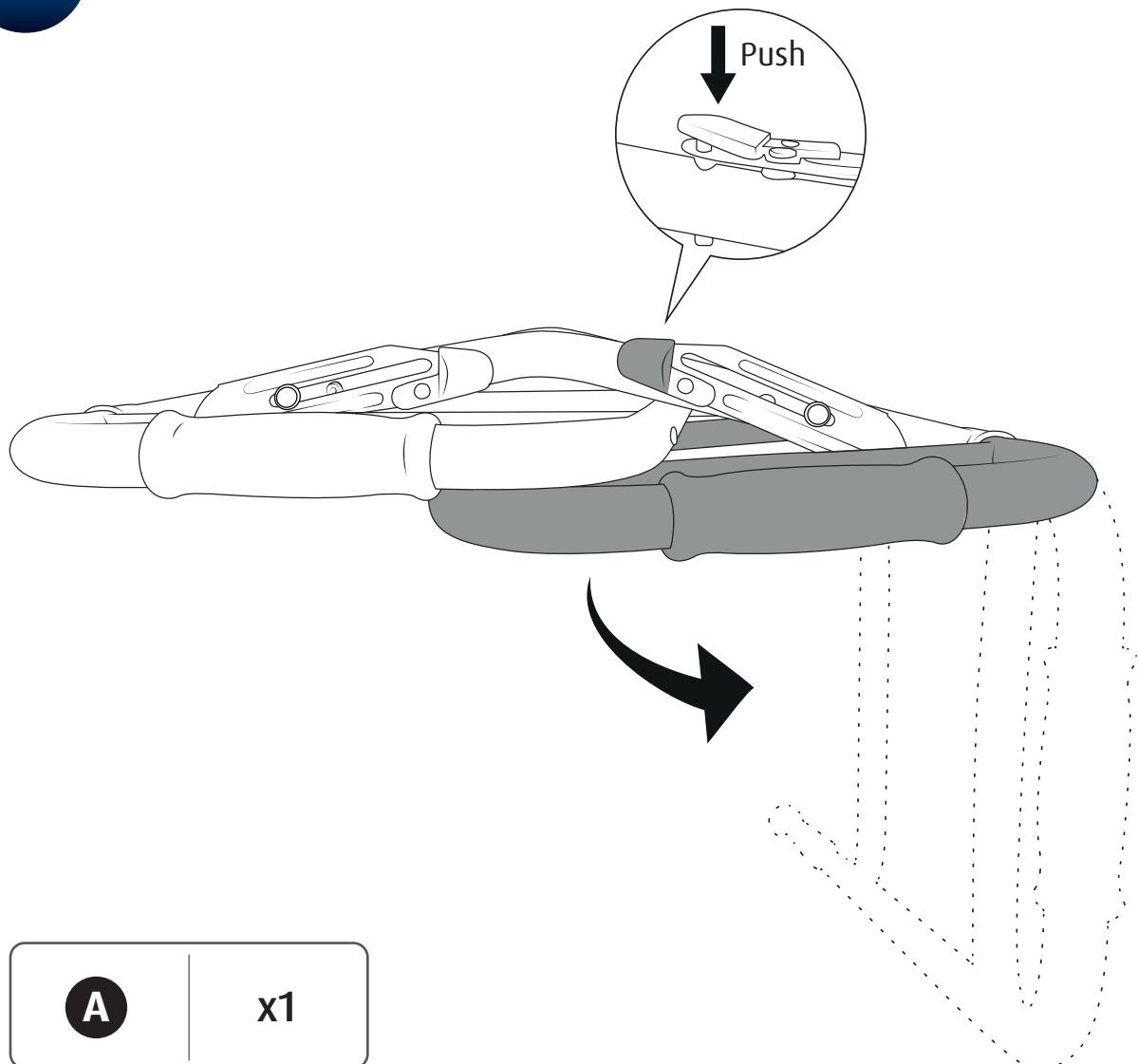A

x1

2

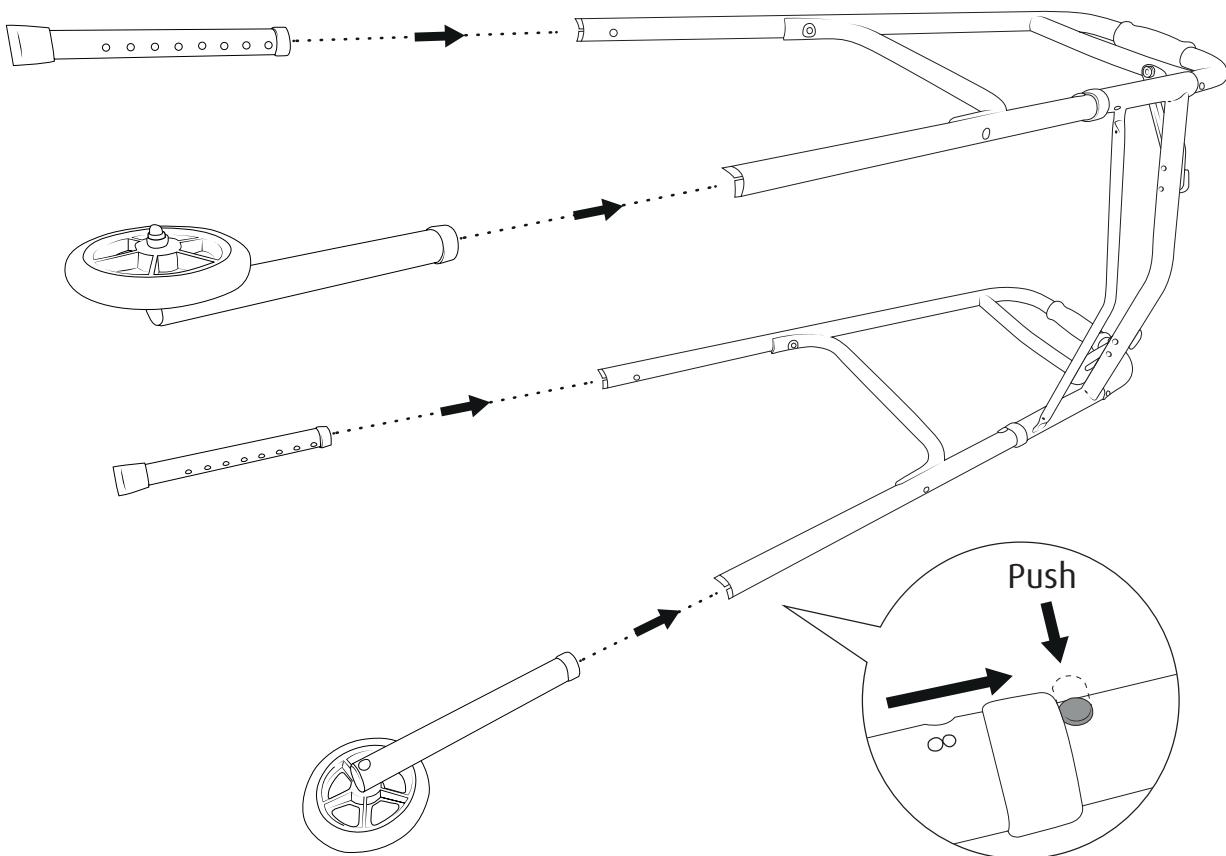| | |
|---|----|
| A | x1 |
| B | x1 |
| C | x1 |

General guidelines

Take the time to thoroughly read the following instruction and utilize the product accordingly.

Ensure you retain this manual and pass it along when transferring the product.

Please be aware that this summary may not encompass every detail of all variations and deliberated steps. Feel free to reach out to us for additional information and assistance.

Warnings

To prevent the risk of suffocation, keep all parts and packaging materials (such as films, plastic bags, and foam) away from babies and children.

While assembling, keep small parts out of the reach of children. Swallowing or inhaling these parts can be fatal.

It is strictly forbidden to use the product for anything other than its intended use. Children must be supervised by adults when using the product.

Alterations or modifications to the product without authorization can compromise safety.

Do not use the product if you have a disability or are taking medication that may interfere with safe use of this product.

Clean the product with a mild soap or household cleaner. Do not use organic solvents (such as acetone, lacquer thinner, lighter fluid, dry cleaning fluid, petrol or turpentine) that may adversely affect the product.

Try not to rely on the walker in situations where quick movement is necessary.

Notes

Carefully follow the enclosed instructions for assembling the walking frame. Make sure all parts are securely attached and that there are no loose parts.

Adjust the height of the walker so that the handles are at wrist height when you are standing upright. Your elbows should be slightly bent when holding the handles.

Take small steps when turning and be careful when manoeuvring around corners and in tight spaces. Avoid sudden movements.

Take extra care when using the walker on slopes. Lean back slightly when going down and forward when going up to maintain your centre of gravity.

Always follow the advice of your healthcare professional on how best to use the walker in relation to your personal health condition.

Only use authentic Dunimed parts if you need to replace any part of the product.

Store this product at room temperature in a dry, ventilated place to prevent mould growth.

This product is intended for use by one person.

Perform regular maintenance checks on the walker, especially the wheels and locking mechanisms, to ensure they are in good condition. Do not use the walker if any parts are loose or damaged.

Always follow the advice of your healthcare provider on how best to use the walker in relation to your personal health condition.

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